

Ocutears® Hydro+

Instructions for use

Preservative free ophthalmic solution

What is Ocutears® Hydro+ and what it is used for?

Ocutears® Hydro+ is a lubricating ophthalmic solution, based on high concentrated cross-linked sodium hyaluronate (0.4%), formulated for the relief of dry eye symptoms, even in severe cases, ocular discomfort and corneal protection.

This product is sterile and preservative free.

Cross-linked sodium hyaluronate gives the product a high viscoelasticity ensuring a high degree of hydration and permanence on the ocular surface, providing long lasting relief from dry eye symptoms, such as soreness, burning, stinging, sandy or gritty sensation, tired heavy or watery eyes.

Ocutears® Hydro+ is indicated:

- for the relief of dry eye symptoms, ocular discomfort and also for corneal protection as a tear substitute, providing lasting hydration.
- can be administered for ocular lubrication even when using contact lenses.

Ocutears® Hydro+ is useful during and after:

- prolonged exposure to sunlight;
- exposure to air-conditioned rooms;
- prolonged screen use;
- contact with cleansing agents;

Ocutears® Hydro+ could also relieve the post surgery discomfort of the eye.


How to use Ocutears® Hydro+

Administer 1 or 2 drops in each eye one or more times a day or as advised by your doctor or pharmacist.

Wash your hands before opening the container and pay attention to hygiene during handling.

Bottle:


- 1) Before first opening pull and remove the tamper evident band completely.
- 2) Gently squeeze 1 drop into the eye.
- 3) Close firmly the bottle after use.

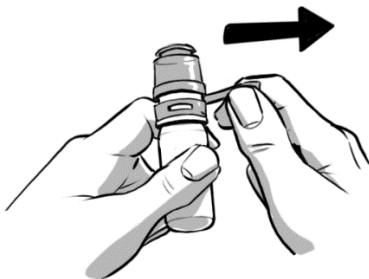

1




2

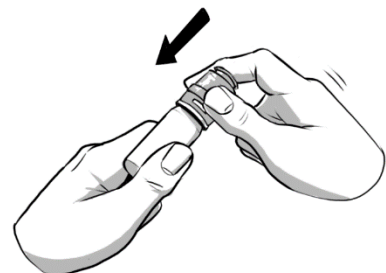

3

UD:

- 1) Take the strip of minicontainers and remove one.
- 2) Open the container by twisting the cap.
- 3) Gently squeeze 1 drop into the eye.
- 4) Discard properly the remaining product in the minicontainer after use.

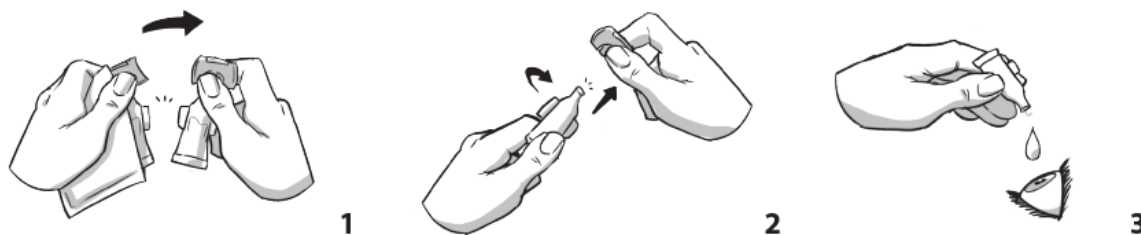

Read the whole leaflet carefully before you start using this product.
 Keep this leaflet; you may need to read it again.
 If you have any further questions, ask your doctor or pharmacist.
 If symptoms persist or you have any side effects please consult your doctor.

Warnings:


- Do not use the product if allergic or hypersensitive to any of the components;
- Use the bottle within 90 days from opening; the single dose container should be used immediately after opening and discarded after use;
- Do not use the product after the expiry date indicated on the pack;
- Do not use the product if the packaging is damaged;
- Keep away from heat, sunlight, moisture and the reach of children;
- Do not ingest;
- Take care that the tip of the container does not come into contact with the eye;
- Store the product below 25°C. Do not freeze;


Composition:

Cross-linked sodium hyaluronate 0.4%, Disodium tetraborate decahydrate, Boric acid, Sodium chloride, Water for injections (WFI).

Presentations: 10 ml bottle (PFMD), 0.35 ml x 15 UD, 0.35 ml x 5 UD (sample)





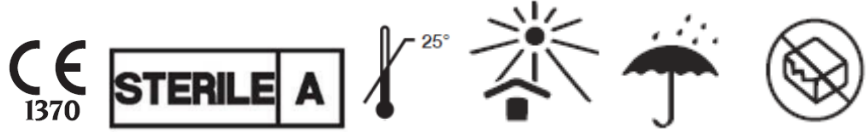






For further information please contact your doctor or your local Santen Office at www.santen.eu

 OFFHEALTH S.p.A., Via Giovanni Paisiello 10, 50144 Florence, Italy



This leaflet was revised in December 2020



										
Manufacturing date	Manufacturer	Store below 25°C	Keep away from sunlight	Sterilized using aseptic processing techniques	Batch code	Use by date	Consult instructions for use	Do not re-use	Do not use if package is damaged	Keep dry


OSD Carton

Ocutears® Hydro+

For Dry Eye
HA 0.4% Lasting Hydration
Preservative free artificial tear
10 ml

Lubricating ophthalmic solution with cross-linked sodium hyaluronate, for the relief of dry eye symptoms, even in severe cases, ocular discomfort and corneal protection.

Composition: Cross-linked sodium hyaluronate 0.4%, Disodium tetraborate decahydrate, Boric acid, Sodium chloride, WFI.

 OFFHEALTH S.p.A., Via Giovanni Paisiello 10, 50144 Florence, Italy



 REF 0570H

Santen

EYE HEALTH SPECIALISTS



EAN13

Rev. 05 of 12/2020

OSD label

Ocutears® Hydro+

HA 0.4% Lasting Hydration
Preservative free artificial tear
10 ml

 OFFHEALTH S.p.A., Florence, Italy



Santen


UD Carton SALE

Ocutears® Hydro+

For Dry Eye
HA 0.4% Lasting Hydration
Preservative free artificial tear

Lubricating ophthalmic solution with cross-linked sodium hyaluronate, formulated for the relief of dry eye symptoms, even in severe cases, ocular discomfort and corneal protection.

Composition: Cross-linked sodium hyaluronate 0.40%, Disodium tetraborate decahydrate, Boric acid, Sodium chloride, WFI.

 OFFHEALTH S.p.A., Via Giovanni Paisiello 10, 50144 Florence, Italy



STERILE A



REF 0520H

LOT



Santen

EAN13

15
Single-dose
Containers
of 0.35 ml
Preservative free

EYE HEALTH SPECIALISTS

Rev. 05 of 12/2020

UD Carton SAMPLE

Ocutears® Hydro+

For Dry Eye
HA 0.4% Lasting Hydration
Preservative free artificial tear

Lubricating ophthalmic solution with cross-linked sodium hyaluronate, for the relief of dry eye symptoms, even in severe cases, ocular discomfort and corneal protection.

Composition: Cross-linked sodium hyaluronate 0.40%, Disodium tetraborate decahydrate, Boric acid, Sodium chloride, WFI.

 OFFHEALTH S.p.A., Via Giovanni Paisiello 10, 50144 Florence, Italy


1370

STERILE A



REF 0530H

LOT



Santen

EAN13

5
Single-dose
Containers
of 0.35 ml
Preservative free

EYE HEALTH SPECIALISTS
FREE SAMPLE NOT FOR SALE

Rev. 05 of 12/2020