



PURALID®
LIPOGEL

P **U** **R** **A** **L** **I** **D**
P **U** **R** **A** **L** **I** **D**
P **U** **R** **A** **L** **I** **D**
P **U** **R** **A** **L** **I** **D**
P **U** **R** **A** **L** **I** **D**
P **U** **R** **A** **L** **I** **D**
P **U** **R** **A** **L** **I** **D**

Instructions for Use

Puralid® Lipogel for use in Blepharitis or Blepharoconjunctivitis

What is Puralid® Lipogel?

Puralid® Lipogel is an ophthalmic product which soothes and cleanses the eyelids and the periorcular area. It contains Liposomes, Tea Tree Oil, Vitamin A, Xanthan gum, Ferulic acid, Vitamin ETPGS and Alfa Bisabolol.

Liposomes, formed by a double layer of phospholipids, are known for their soothing and emollient properties, as well as the ability to effectively deliver the working ingredients.

Tea Tree Oil, extracted from Melaleuca alternifolia, is known for its antimicrobial properties.

Vitamin A strengthens and maintains the skin by regulating epithelial function. It is also an antioxidant which, together with Xanthan Gum, Ferulic acid and Vitamin E TPGS, acts in synergy with liposomes to protect the eyelid surface from the oxidative action of free radicals and UV rays.

Alfa Bisabolol is the main component of chamomile essential oil. It has soothing properties, reduces redness and inflammation, even in case of particularly delicate skin. It is characterised by good skin tolerability.

What is Puralid® Lipogel used for?

Puralid® Lipogel is used to soften and remove flaking residue on the eyelid during inflammatory events like blepharitis or blepharoconjunctivitis.


How often should I use Puralid® Lipogel?

Use the product twice a day, or more if advised by your doctor.


How to use Puralid® Lipogel

Before first use, hold the bottle upright and push the pump firmly 3/5 times to load the dispensing system, until it delivers the gel.


1. Wash your hands carefully.

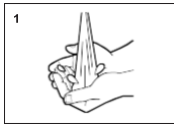


2. Dispense a small amount of Puralid® Lipogel on your fingertips.

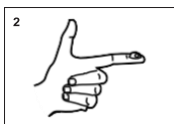

3. Gently spread it on the eyelids with circular movements.

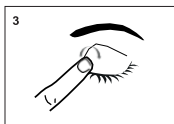

4. Leave on for a few minutes. Wipe away any flakes with a clean tissue or cotton pad.

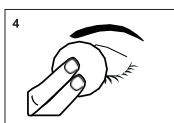

Read the whole leaflet carefully before you start using this product.

Keep this leaflet; you may need to read it again.

If you have any further questions, ask your doctor or pharmacist.

What are the possible side effects?

IF YOU NOTICE ANY SIDE EFFECTS PLEASE INFORM YOUR DOCTOR OR PHARMACIST.

How to store Puralid® Lipogel

- Do not store above 25°C;
- Do not freeze;
- Keep away from heat, moisture, direct sunlight;
- Keep out of reach and sight of children.

Composition:

Phospholipids S80 (by NIOlip), Xanthan gum, Bisabolol, Tea Tree Oil, Vitamin E TPGS, Ferulic Acid, Vitamin A Palmitate, EDTA disodium dihydrate, Potassium Sorbate, Sodium Benzoate, Sodium Hydroxymethylglycinate, Citric acid monohydrate, Potassium Citrate Tribasic Monohydrate, Distilled water.

Warnings

- External use only;
- Do not use the product if you are hypersensitive to any of its components;
- Do not use the product after the expiry date indicated on the pack;
- Do not use the product if the packaging is damaged;
- Use the product within 8 weeks from first dispensing (15ml bottle);
- Use the product within 24 hours from first dispensing (1ml mini-container).

OFFHEALTH SpA
Via G. Paisiello 10
50144 Florence
Italy



Presentations:

Commercial - 15 ml bottle
Sample - strip of 5 mini-containers of 1ml
For further information please contact your doctor or your local Santen Office at www.santen.eu

Santen

This leaflet was validated in December 2020

Manufacturer	Manufacturing date	Lot number	Expiry date	Read the instruction for use	Do not use if the package is damaged	Keep dry	Store below 25 °C	Protect from sunlight

Carton sale

Carton sample

PURALID® Lipogel
For use in blepharitis

Cleanses, soothes & relieves
inflamed eyelids

For use in Blepharitis or
Blepharoconjunctivitis

Puralid® Lipogel is an ophthalmic
product which soothes and cleanses
the eyelids and the periocular area.

Softens and removes flaking residue
on the eyelid during inflammatory
events like blepharitis or
blepharoconjunctivitis.

Read the package leaflet before use
Do not store above 25°C - Do not
freeze


Keep away from heat, moisture,
direct sunlight

Keep out of reach and sight of
children

Composition: Phospholipids S80
(by NIOlip), Xanthan gum,
Bisabolol, Tea Tree Oil, Vitamin E
TPGS, Ferulic Acid, Vitamin A
Palmitate, EDTA disodium dihydrate,
Potassium Sorbate, Sodium
Benzoate, Sodium Hydroxymethyl-
glycinate, Citric acid monohydrate,
Potassium Citrate Tribasic
Monohydrate, Distilled water.

 OFFHEALTH SpA
Via G. Paisiello 10
50144 Florence
Italy

Importer/distributor:

 033OH



Santen

LOT



15 ml
Eye health specialists

PURALID® Lipogel
For use in blepharitis

Cleanses, soothes & relieves
inflamed eyelids

For use in Blepharitis or
Blepharoconjunctivitis

Puralid® Lipogel is an ophthalmic
product which soothes and cleanses
the eyelids and the periocular area.

Softens and removes flaking residue
on the eyelid during inflammatory
events like blepharitis or
blepharoconjunctivitis.

Read the package leaflet before use
Do not store above 25°C - Do not
freeze


Keep away from heat, moisture,
direct sunlight

Keep out of reach and sight of
children

Composition: Phospholipids S80
(by NIOlip), Xanthan gum,
Bisabolol, Tea Tree Oil, Vitamin E
TPGS, Ferulic Acid, Vitamin A
Palmitate, EDTA disodium dihydrate,
Potassium Sorbate, Sodium
Benzoate, Sodium Hydroxymethyl-
glycinate, Citric acid monohydrate,
Potassium Citrate Tribasic
Monohydrate, Distilled water.

 OFFHEALTH SpA
Via G. Paisiello 10
50144 Florence
Italy

Importer/distributor:

 034OH



Santen


LOT



1 ml x 5
Eye health specialists
Free sample not for sale

Label

PURALID® Lipogel
For use in blepharitis
Cleanses, soothes & relieves
inflamed eyelids

 OFFHEALTH SpA
Italy



Santen

LOT

